


**WECK**



Weck  
**Hem-o-lok**  
System Procedure Guide

**Teleflex**

## Vessel/Tissue Bundle Size Range

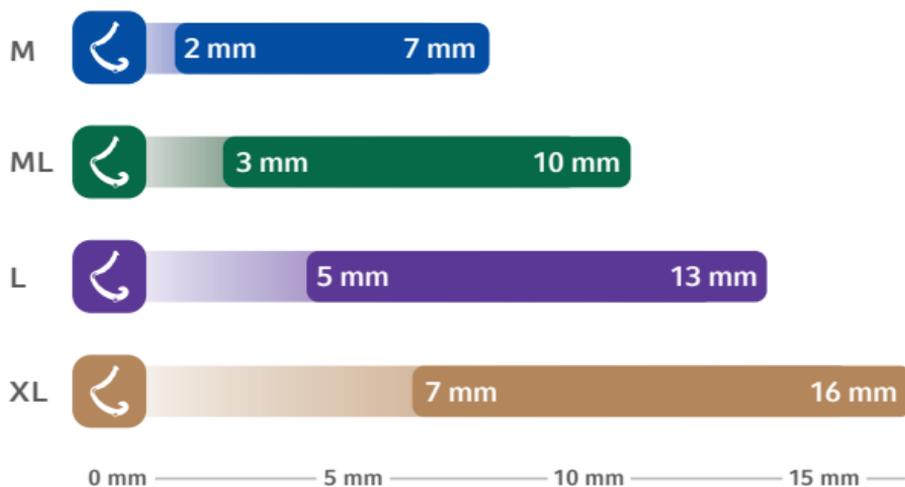

### HEM-O-LOK POLYMER LIGATING CLIPS

- 544220 Medium
- 544230 Medium-Large
- 544240 Large
- 544250 Extra-Large



# General-Bariatric Surgery

## Anterior Colorectal Resection

Distal colon and proximal rectum removal

### STRUCTURES LIGATED

- Inferior mesenteric vessels or branches
- Superior haemorrhoidal vessels
- Middle haemorrhoidal vessel

### COMMON CLIP SIZES



## Appendectomy

Inflamed appendix removal

### STRUCTURES LIGATED

- Appendiceal mesentery and vessels
- Appendiceal stump

### COMMON CLIP SIZES



## Cholecystectomy

Gall bladder removal

### STRUCTURES LIGATED

- Cystic artery
- Cystic duct

### COMMON CLIP SIZES



## Gastrectomy

Removal of part or all of stomach

### STRUCTURES LIGATED

- Gastric arteries
- Gastroepiploic artery
- Gastrocolic vessels
- Gastrosplenic vessels

### COMMON CLIP SIZES



## Left Colectomy

Removal of left half of colon

### STRUCTURES LIGATED

- Gastrocolic ligament
- Left colic vessels or branches
- Inferior mesenteric vessels or branches

### COMMON CLIP SIZES



## Nissen Fundoplication

Treatment for hiatus hernia, gastric juice reflux

### STRUCTURES LIGATED

- Short gastric vessels

### COMMON CLIP SIZES



## Right Colectomy

Removal of right half of colon

### STRUCTURES LIGATED

- Gastrocolic ligament
- Ileocolic vessels or branches
- Right colic vessels or branches
- Middle colic vessels or branches

### COMMON CLIP SIZES



## Subtotal Colectomy

Removal of entire colon

### STRUCTURES LIGATED

- Ileocolic vessels
- Right colic vessels
- Middle colic vessels
- Left colic vessels
- Sigmoid vessels
- Superior haemorrhoidal vessels

### COMMON CLIP SIZES



# Urologic Surgery

## Nephrectomy

Kidney removal

### STRUCTURES LIGATED

- Renal artery\*
- Renal vein
- Adrenal vein
- Lumbar vein
- Gonadal vein
- Ureter

### COMMON CLIP SIZES



ML



L



XL

\* Hem-o-lok Ligating Clips are contraindicated for use in ligating the renal artery during laparoscopic donor nephrectomies

## Prostatectomy

Prostate gland removal

### STRUCTURES LIGATED

- Vas deferens
- Prostatalic vessels
- Dorsal vein complex
- Puboprostatic ligaments
- Preprostatic veins

### COMMON CLIP SIZES



ML



L



XL

# Gynecological Surgery

## Hysterectomy

Uterus and ovary removal

### STRUCTURES LIGATED

- Round ligament
- Ovarian ligament
- Uterine artery
- Infundibular pelvic ligament
- Fallopian tubes\*\*

### COMMON CLIP SIZES



ML



L



XL

\*\* Hem-o-lok Ligating Clips are not intended for use as a fallopian contraceptive tubal occlusion device.

## Additional Procedures

### Adrenalectomy

### Splenectomy

# ORDERING INFORMATION

## Weck Hem-o-lok Polymer Locking Ligation System

CLIP



**M**

**544220**

**Hem-o-lok Medium Polymer Clips**

6 clips/cartridge • 14 cartridges per box, sterile

OPEN MANUAL LOAD  
APPLIERS/REMOVERS

ENDOSCOPIC MANUAL LOAD  
APPLIERS/REMOVERS

**544113** 20 cm curved  
**544115** 27 cm curved  
**544114** 27 cm right-angle (70°)  
**544123** 27 cm remover  
(for M, ML, L)

**Endo5 M Applier, 5mm diameter**  
**544945T 34cm Medium applier,**  
**take-apart**

CLIP



**ML**

**544230**

**Hem-o-lok Medium-Large Polymer Clips**

6 clips/cartridge • 14 cartridges per box, sterile

OPEN MANUAL LOAD  
APPLIERS/REMOVERS

ENDOSCOPIC MANUAL LOAD  
APPLIERS/REMOVERS

AUTOMATIC ENDOSCOPIC  
APPLIERS

**544170** 20 cm curved  
**544171** 27 cm curved  
**544172** 27 cm right-angle (70°)  
**544123** 27 cm remover  
(for M, ML, L)

**Endo5 ML, 5 mm Diameter**  
**544965T** 34 cm applier TAP  
**544965T20** 45 cm applier TAP 20°  
**544965L** 45 cm applier long  
**544121T** 32 cm remover, TAP  
(for ML, L)  
**544121L** 45 cm remover long  
(for ML, L)

**AE05ML** 35 cm applier  
15 ML clips per applier,  
3 appliers/box,  
sterile, disposable

CLIP



**L**

**544240**

**Hem-o-lok Large Polymer Clips**

6 clips/cartridge • 14 cartridges per box, sterile

OPEN MANUAL LOAD  
APPLIERS/REMOVERS

ENDOSCOPIC MANUAL LOAD  
APPLIERS/REMOVERS

AUTOMATIC ENDOSCOPIC  
APPLIERS

**544180** 20 cm curved  
**544182** 25 cm right-angle (70°),  
narrow shaft  
**544181** 27 cm curved  
**544179** 27 cm right-angle (70°)  
**544123** 27 cm remover  
(for M, ML, L)  
**544124** 27 cm remover  
(for L, XL)

**Endo10 L**  
**544995T** 34 cm, 10 mm applier TAP  
**544995T20** 34 cm, 10 mm applier TAP 20°  
**544995L** 45 cm, 10 mm applier long  
**544121T** 32 cm, 5 mm remover TAP  
(for ML, L)  
**544121L** 45 cm, 5 mm remover long  
(for ML, L)  
**544130T** 33 cm, 10 mm remover TAP  
(for L, XL)  
**544130T20** 33 cm, 10 mm remover TAP 20°  
(for L, XL)  
**544130L** 45 cm, 10 mm remover long  
(for L, XL)



## ORDERING INFORMATION (cont.)

### Weck Hem-o-lok Polymer Locking Ligation System

CLIP



**XL**

**544250**

**Hem-o-lok Extra-Large Polymer Clips**

6 clips/cartridge • 14 cartridges per box, sterile

OPEN MANUAL LOAD APPLIERS/REMOVERS	ENDOSCOPIC MANUAL LOAD APPLIERS/REMOVERS	AUTOMATIC ENDOSCOPIC APPLIERS
<p><b>544191</b> 27 cm curved</p> <p><b>544192</b> 27 cm right-angle (70°)</p> <p><b>544124</b> 27 cm remover (for L, XL)</p>	<p><b>Endo10 XL, 10 mm Diameter</b></p> <p><b>544990T</b> 34 cm applicator TAP</p> <p><b>544990T20</b> 34 cm applicator TAP 20°</p> <p><b>544990L</b> 45 cm applicator long</p> <p><b>544130T</b> 33 cm remover TAP (for L, XL)</p> <p><b>544130T20</b> 33 cm remover TAP 20° (for L, XL)</p> <p><b>544130L</b> 45 cm remover long (for L, XL)</p>	
<p><b>STORAGE TRAY</b></p>	<p><b>544500</b> capacity: 3 endoscopic 32 cm applicators</p>	

TAP = take-a-part

Hem-o-lok Ligating Clips are not intended for use as a fallopian contraceptive tubal occlusion device.

Hem-o-lok Ligating Clips are contraindicated for use in ligating the renal artery during laparoscopic donor nephrectomies.

For more information visit  
[teleflex.link/surgery-emea](http://teleflex.link/surgery-emea)



Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives.

We apply purpose-driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit [teleflex.com](http://teleflex.com).

Teleflex is the home of Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüschi, UroLift and Weck – trusted brands united by a common sense of purpose.

### **Corporate Office**

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

### **Regional Offices**

**United States:** Phone +1 919 544 8000, Toll Free 866 246 6990, [cs@teleflex.com](mailto:cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**Latin America:** Phone +1 919 433 4999, [la.cs@teleflex.com](mailto:la.cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**International:** Phone +353 (0)9 06 46 08 00, [orders.intl@teleflex.com](mailto:orders.intl@teleflex.com), Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Teleflex, the Teleflex logo, Arrow, Deknatel, Hem-o-lok, Hudson RCI, LMA, Pilling, Rüschi, UroLift and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 07/2020

© 2020 Teleflex Incorporated. All rights reserved.

MCI-2020-0497-EN · REV 0 · 07 20 PDF